

Fitting & Care Instructions

Model: DCT-QF37

Instructions on putting on brace:



Step 1: Open brace and lay flat on a table



Step 2: Remove lateral panels



Step 3: On right and left side lift and detach the overlapping sizing extension



Step 4: When completely detached your brace will look like the above image



Step 5: Find your size (SM-3XL) and reattach the overlapping sizing system to the correct size



Step 6: Repeat step 5 to the opposite side so both sides are the same size. Reattach the lateral panel to both sides



Step 7: Once properly sized center the back brace in the middle of your back (logo at the top) and wrap around your waist.



Step 8: Wrap the side wings right side overlapping the left side and pull the pull strings to your desired compression

WARRANTY

The DCT-QF37 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.

CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.



TREND MEDICAL