

For video instructions hold camera phone up to code.



Fitting & Care Instructions

Model: DCT-5657

Instructions on putting on brace:



Step 1: Open brace and lay flat on a table. Detach side wings from right and left to find your waist circumference and size.



Step 2: Loosen shoulder straps and slide on like a book bag.



Step 3: Wrap side wings around waist and Velcro closed around your



Step 4: Tighten shoulder straps by pulling down on straps.



Step 5: Tighten waist by pulling on rubber pull tabs to desired support.



Step 6: Locate chest support buckle and adjust height by detaching from shoulder straps



Step 7: Once correct height is found tighten by buckling and pulling on the straps.



Step 8: Find rubber pull tabs in a heart shape. Pull to desired shoulder extension support.

Care Instructions

Hand wash with a light detergent and air dry



TREND MEDICAL